

ASSEMBLY INSTRUCTIONS FOR

Tamper Evident Tip Cap for Colored Oral Dispensers (#17286)



Step 1. Press cap onto the tip of the dispenser and press firmly until it clicks into place.



Step 2. To remove the cap, pull the red cap covering.



Step 3. The plastic sleeve is left behind when the cap is removed to indicate the syringe tip has been uncovered or tampering has occurred. Remove the white cap prior to use.

NOTE: Before dispensing, be sure to remove the plastic sleeve and white cap.

Call Free 1.800.848.1633 • Fax Free 1.800.447.2923
Web: www.GoHCL.com • Email: hcl@GoHCL.com