


Instructions for the Tamper Evident Tip Cap for Luer Lock Syringes (HCL #17360/17361)



1. Place the Tamper Evident Tip Cap on the tip of the syringe and press down firmly to lock it on the syringe tip.

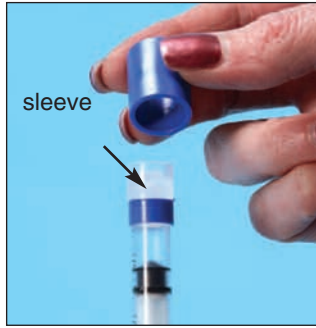

2. To remove the tamper cap, pull the cap covering. The plastic sleeve and white cap will be left behind. This is what it would look like if the cap has been removed to tamper with the syringe.



3. Before using the syringe, remove the cap and sleeve.



© Health Care Logistics, Inc. 2007

HCL-INTL is a registered trademark of Health Care Logistics, Inc.

010717360

Call Free 1 800 848 1633 or 888 HCL INTL® (425 4685) • Fax Free 1 800 447 2923

International Call 740 477 1686 • International Fax 740 477 2923 • Web site: www.HealthCareLogistics.com • E-mail: HCL@HealthCareLogistics.com