

CLEANING GUIDE FOR

Lock-to-Pole IV Lock Boxes (#17938 & 17958)

Wipe surfaces with facility-approved disinfectant, taking care to avoid electronic components. Allow to air dry per dwell time provided on disinfectant label.

Warnings:

Disinfectant should be compatible with materials listed below. Some disinfectants are not compatible with polycarbonate. Discontinue use if signs of damage occur.

Materials:

Polycarbonate
POM (acetal)
TPE (Thermoplastic Elastomer)
Polypropylene

Suggested Disinfectant:

19954, PREempt™ One-Step Surface Cleaner and Disinfectant Wipes, Canister
19954-31, PREempt™ One-Step Surface Cleaner and Disinfectant Wipes, Case