


ASSEMBLY INSTRUCTIONS FOR

Double IV Lock-to-Pole IV Lock Box (#18638)

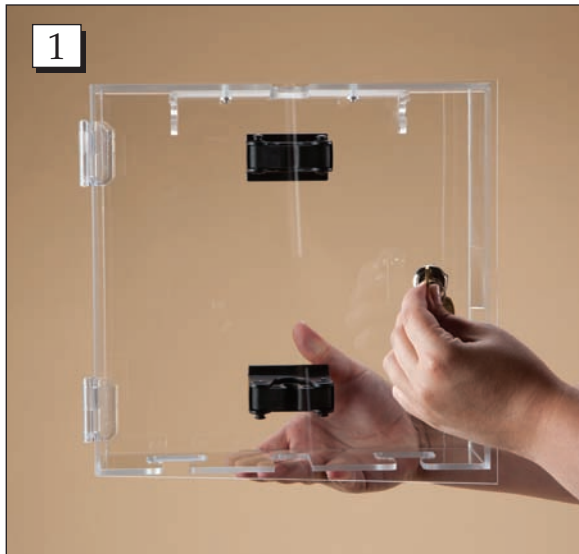

Step 1. Open the door of the IV box.



Step 2. Align the clamps on the inside of the box with your pole. Holding the box with your four fingers and the pole with your thumb, press firmly until the clamps wrap around the pole.



Step 3. Hang the IV bags on the hooks.



Step 4. Lock the IV box door.

Call Free 1.800.848.1633 • Fax Free 1.800.447.2923
Web: GoHCL.com • Email: hcl@GoHCL.com