

## Frequently Asked Questions:

### *How do I clean the boxes?*

You can use Sani-Cloth Germicidal Disposable Wipes (18578) after each use to help reduce preventable infections.

### *Why are there two vials of Humalog?*

Humalog is used more often than the other two types of insulin.

### *Do I have to label the vials with any auxiliary labels?*

No, the box comes with the "High Alert" and Type of insulin labels on it already.

### *Do I need to open the box to access the drug to dispense it?*

No, just remove the vial cap, swab the vial injection port with alcohol, and withdraw the desired amount.

### *How do I know when the insulin inside the ISSI box is expiring?*

Most institutions keep a log and exchange all the vials at one time, by either having another set in the pharmacy ready, or exchange the vials at a nursing unit.

- \*\*\*The box should not go in a contact isolation room. If you should choose to use for patient on Isolation precautions that box should be used for that patient only and disposed of.
- \*\*\*The vials can be rotated to get a better view of barcode, expiration date, lot number, and drug name.
- \*\*\*The HumuLIN and HumaLOG are located at different parts of the box to avoid medication errors.
- \*\*\*The box is large enough so that it cannot be accidentally lost or left in the pocket.
- \*\*\*The box can fit in a nursing med cart or ADC (Automatic Dispensing Cabinet).



Additional questions? email [ISSI@gohcl.com](mailto:ISSI@gohcl.com)

**Health Care Logistics<sup>®</sup>**  
INC.

**GoHCL.com • 1.800.848.1633**