

INSTRUCTIONS FOR Pill Puncher (#7537)



Step 1. For extra-tough blister backings, first perforate the backing using the wheel on the handle of the puncher.



Step 2. Place the blister pack foil side down in the cup part of the puncher.



Step 3. Squeeze the tongs of the puncher together firmly to eject the tablet into the cup.



Step 4. Remove the empty blister and then empty the tablet from the cup into a med cup.



Call Free: 1.800.848.1633 • Fax Free: 1.800.447.2923
GoHCL.com • hcl@GoHCL.com