

Instructions for the Single Tube Transport Bubble Bag (HCL #7618-10)



1) Place the container into the Transport Bubble Bag.



2) Peel the tape away from the adhesive backing.



3) Fold the top flap once to seal the bag.



4) Contents are now protected and ready for transport.



Call Free 1 800 848 1633 or 888 HCL INTL (425 4685) • Fax Free 1 800 447 2923

International Call 614 477 1686 • International Fax 614 477 2923 • Web site <http://www.hcl-intl.com> • E-mail: HCL@hcl-intl.com