

Bag Spikes with SmartSite®



Bag Spike with SmartSite

Sterile SmartSite needle-free bag access spike.

Specifications

- This device does not contain DEHP in the fluid path.
- Individually packaged.



Contraindications:

None when used as described in product insert.



- Single use only. Do not resterilize.
- Fluid path is STERILE and NONPYROGENIC. Discard if packaging is not intact or protector caps are missing or loose.



Bag Spike with Check Valve and SmartSite

Sterile SmartSite needle-free bag access spike with check valve.



Latex-Free

Instructions for use

Use Aseptic Technique

Note: The valve port is two-way and luer-locking.

- 1** Remove protective cap on spike.
- 2** Insert spike into fluid container. Suspend fluid container.
- 3** Swab valve with preferred antiseptic prior to accessing valve port.
- 4** Attach syringe to valve port.
- 5** Withdraw desired amount of fluid. Detach syringe.
- 6** Repeat steps 2 through 4 for subsequent fluid withdrawals.

NOTE: The check valve prevents medication additions to the solution container.



			Order No. REF.
Bag Spike with SmartSite	1	20	H938 93 5
Bag Spike with SmartSite & Check Valve	1	20	H938 94 5

Vial Adapter with SmartSite®



Sterile SmartSite needle-free vial access device for multiple withdrawals or additions of fluids.

Specifications

- Fluid path contains no DEHP, DOP plasticized PVC.
- Individually packaged.



Contraindications:

None when used as described in product insert.



- Single use only. Do not resterilize.
- Fluid path is STERILE and NONPYROGENIC. Discard if packaging is not intact.



Latex-Free

Instructions for use

Use Aseptic Technique

Note: The valve port is two-way and luer-locking.

- 1** Swab septum with preferred antiseptic prior to accessing valve port.
- 2** Attach access device to vial.
- 3** Connect syringe to needle-free valve port.
- 4** Withdraw or add desired fluid volume.



			Order No. REF.
Vial Adapter with SmartSite	1	20	H938 92 5



Corporate Headquarters - USA

Baxa Corporation
13760 E. Arapahoe Rd.
Englewood, CO 80112-3903
tel: 303-690-4204
fax: 303-690-4804
toll free: **800-567-BAXA (2292)**
E-mail: baxamail@baxa.com
www.baxa.com

Registered Agent - United Kingdom

Baxa Limited
Glendale Park
Fernbank Road
North Ascot
Berkshire SL5 8JB
tel: +(44) 1344 890916
fax: +(44) 1344 890917
E-mail: enquiries@baxa.co.uk

Canada

Baxa Canada
210 Cochrane Dr. Unit #2
Markham, Ontario
L3R 8E6
tel: 905-946-0319
fax: 905-946-1710
toll free: 800-357-BAXA (2292)
E-mail: baxamail@baxa.ca

To find a local Baxa agent, visit our website at www.baxa.com or call our offices directly.