

Instructions for the Bin Cart (HCL #1436)



Before assembling your Bin Rack, please familiarize yourself with all the parts, and check to make sure that you have all the parts as listed below.

Note: We recommend using a $\frac{3}{8}$ " drive reversible ratchet, $\frac{7}{16}$ " open end wrench, screwdriver, and appropriate sockets for assembly of unit.

- A** - (2) Vertical Bars
- B** - (12) Crossbars
- C** - (48) Hex Head Sheet Metal Screws
- D** - (1) Work Top
- E** - (4) Work Top Braces
- F** - (1) Handle
- G** - (4) Horizontal Braces
- H** - (2) Base Assemblies
- I** - (4) Swivel Wheel
- J** - (16) Hex Head Bolts, Nuts, and Washers
- K** - (16) Slotted Head Screws, Nuts, and Washers



1. Attach a vertical bar onto the first base unit.

Your small friend, indeed[®]
Health Care Logistics[®]
INC. 

Sheet#1436
Rev.010102

Call Free 1 800 848 1633 or 888 HCL INTL[®] (425 4685) • Fax Free 1 800 447 2923

International Call 740 477 1686 • International Fax 740 477 2923 • Web site <http://www.hcl-intl.com> • E-mail: HCL@hcl-intl.com

Instructions for the Bin Cart (HCL #1422)



2. Screw two screws to keep the vertical bar in place.



3. Attach the other vertical bar to the second base unit.



4. Place the first horizontal bar in between the two base units and the vertical bars.


Call Free 1 800 848 1633 or 888 HCL INTL® (425 4685) • Fax Free 1 800 447 2923

International Call 740 477 1686 • International Fax 740 477 2923 • Web site <http://www.hcl-intl.com> • E-mail: HCL@hcl-intl.com

Instructions for the Bin Cart (HCL #1422)



5. Screw the horizontal bar onto the vertical bar.

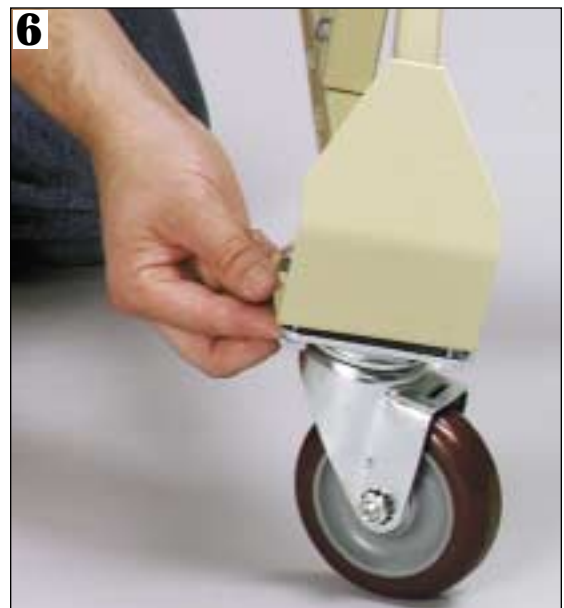

6. Now attach the swivel wheels to the bottom of the base units by simply screwing in the screws.



7. To make sure the cart is secure, attach a horizontal brace from the first horizontal bar to the wheel of the base unit.

Call Free 1 800 848 1633 or 888 HCL INTL® (425 4685) • Fax Free 1 800 447 2923

International Call 740 477 1686 • International Fax 740 477 2923 • Web site <http://www.hcl-intl.com> • E-mail: HCL@hcl-intl.com

Instructions for the Bin Cart (HCL #1422)



8. Skip every other hole until 12 horizontal bars are in place. The very top hole will be vacant.



9. Attach four worktop braces to the cart to hold the work top in place.



10. Attach a handle at one end of the cart, this allows the cart to be pushed or pulled.



Call Free 1 800 848 1633 or 888 HCL INTL® (425 4685) • Fax Free 1 800 447 2923

International Call 740 477 1686 • International Fax 740 477 2923 • Web site <http://www.hcl-intl.com> • E-mail: HCL@hcl-intl.com

Instructions for the Bin Cart (HCL #1436)



11. Place the worktop onto the cart and attach the braces to securely keep it in place.

Congratulations you have now finished assembling your Bin Cart.



Call Free 1 800 848 1633 or 888 HCL INTL® (425 4685) • Fax Free 1 800 447 2923

International Call 740 477 1686 • International Fax 740 477 2923 • Web site <http://www.hcl-intl.com> • E-mail: HCL@hcl-intl.com