#1714 Security Bag for Waterloo Crash Cart Box Instructions



1.Insert the crash cart box into the opening of the 2.Pull the paper backing off the edge of the bag crash cart bag.



to expose the adhesive tape.



3. Seal the bag by folding over the edge of the bag.



4. To open the bag, remove from the cart drawer and tear at the center or the end perforation. Pull one half to the left and the other half to the right.









© Health Care Logistics, Inc. 1999 HCL-INTL is a registered trademark of Health Care Logistics, Inc. Sheet #1714