

ASSEMBLY INSTRUCTIONS FOR

Refrigerator Box (HCL #17508, 17524, 17525, 17526)




Step 1. Place hand inside box and hold into place while placing retention plate on top of refrigerator rack.



Step 2. Thread screw from the inside of the box through the rack into the retention plate - (trap refrigerator rack between box and retention plate).

Retention Plate (with 4 threaded holes
- no nuts required.)

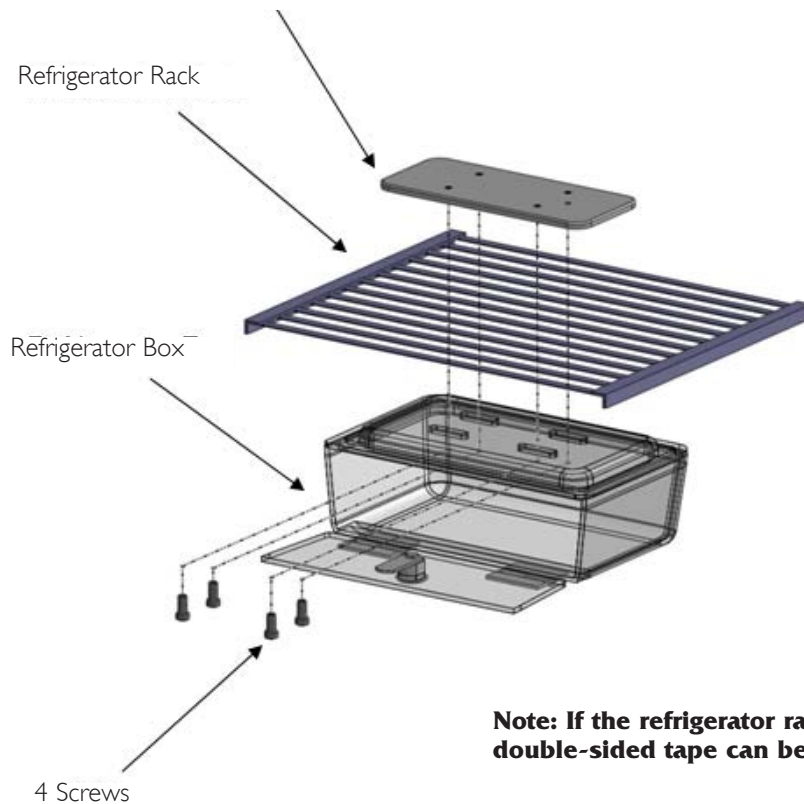

**Note: If the refrigerator rack is solid,
double-sided tape can be used for assembly.**

© Health Care Logistics, Inc. 2008 • Printed in the U.S.A.

Call Free 1.800.848.1633 or 1.888.HCL-INTL Fax Free 1.800.447.2923
Website: www.HealthCareLogistics.com • Email: hcl@HealthCareLogistics.com