

THE MINI MAT™

FOR STEP STOOL APPLICATION

(888)MD-FLUID

17716

DIRECTIONS FOR USE

- 1) Remove *THE MINI MAT*™ from packaging and determine the proper position on step stool (foot stool, riser, etc.). The surface where the mat will lay **MUST** be completely dry and clean prior to definitive placement. Be sure all wording on the mat is facing outward so it can be read while facing the surgical table. **IMPORTANT NOTE:** Make sure mat will not overhang on the sides of step stool in order to prevent any chance of tripping. Do not use mat if overhang is noticed.
- 2) Remove green adhesive strip tab on underside of *THE MINI MAT*™ to expose adhesive and place mat on step stool. Apply firm pressure to ensure adhesive strip is secure to step stool. **IMPORTANT NOTE:** Adhesive strip must always be used when using *THE MINI MAT*™ to prevent mat slippage as a result of shear and other potential forces.
- 3) *THE MINI MAT*™ is now ready for use. **IMPORTANT NOTE:** If mat is repositioned during use, the adhesive strip will not function properly and mat should be discarded.
- 4) At conclusion of procedure, lift *THE MINI MAT*™ off of step stool and discard mat with all other contaminated disposables, as per hospital regulations and guidelines. **IMPORTANT NOTE: NEVER ATTEMPT TO REUSE MAT.**

Note: *THE MINI MAT*™ is not designed to absorb fluids, but rather channel fluids away from the surgeon/assistants standing on the mat.

CAUTION: Do not use mat on uneven, textured or non-skid surfaces as slippage and injury may result.

THE MINI MAT™
is a One-Time Use, Disposable Product!

It **cannot** be cleaned and should never be re-used to ensure avoidance of cross-contamination throughout the operating room.

FLAGSHIP SURGICAL, LLC
16 Mt. Bethel Road, Suite 313
Warren, NJ 07059

www.flagshipsurgical.com



Manufactured by an
ISO 9001:2000 Certified Company



FLAGSHIP SURGICAL



Your small friend, indeed!
Health Care Logistics[®]
INC.