


# INSTRUCTIONS FOR

## Rx Crush™ ENFit™ Tablet Crusher (#19846)



**Step 1.** Draw up solvent into syringe and connect the syringe to the bag.

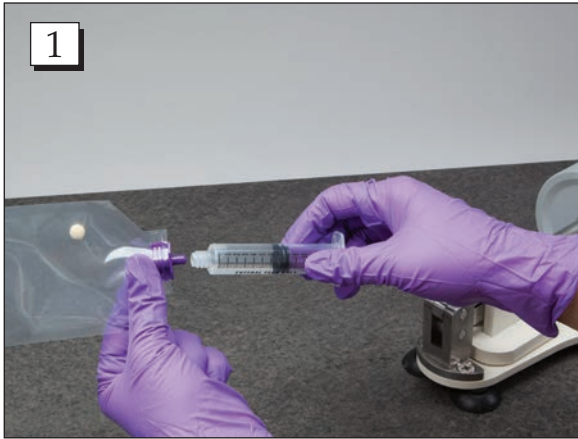

**Step 2.** Crack tablet with the platform.



**Step 3.** Crush tablet with the roller, moving the bag with the attached syringe until desired consistency is reached.



**Step 4.** Inject solvent into the bag to dissolve powder and administer according to drug manufacturer instructions.

Call Free 1.800.848.1633 • Fax Free 1.800.447.2923  
Web: GoHCL.com • Email: hcl@GoHCL.com