

# INSTRUCTIONS FOR HCL® Chemo Spill Kit (19934)

1. Post HD Spill Warning Tent Sign with spill kit label.
2. Don first pair of gloves, apron (ensure sleeves are tucked in between the first and second pair of gloves), second set of gloves, followed by the goggles, mask, shoe covers and bouffant cap.
3. Depending on size of spill, use the Zorb sheets, spill solidifier, and/or Sterile HDClean Two-Step Wipe on the spilled drugs.
4. Use the Bio Scoop and Scraper to pick up spilled drugs.
5. Place items into chemo disposable slider bag and seal shut.
6. Place items into the larger chemotherapy waste bag and/or chemo sharps container along with protective clothing.
7. Leave the caution sign in place until environmental services can clean the area further to ensure everyone's safety.
8. Dispose of all waste in accordance with local, state and federal regulations.