

INSTRUCTIONS FOR HCL® Packout System, 16-Quart (#20365)



Contents Included:

- Transport Cooler, 16-Quart
- Three Cold Packs
- Two Large Corrugated Plastic Dividers
- Packout Instructions

Not Included:

- USB Probe Data Logger with Glycol Bottle Probe
- Cellophane Packing Bags
- Cardboard Box



Step 1.

Place two frozen cold packs in the bottom of the cooler.



Step 2.

Place one corrugated plastic divider over the cold packs.



Step 3.

Place one USB Probe Data Logger that has been chilled to approximately 40°F inside of a cardboard box.



Step 4.

Fill dead space with cellophane packing bags.



Step 5.

Place second corrugated plastic divider over box.



Step 6.

Place the third frozen cold pack on top.



Step 7.

Place an even layer of cellophane packing bags to fill the dead space. Make sure the lid shuts flush with no interference.

Call Free 1.800.848.1633 • Fax Free 1.800.447.2923

Web: GoHCL.com • Email: hcl@GoHCL.com