



INSTRUCTIONS FOR

Stainless Steel Stand for HCL® Dispensers (#20947)

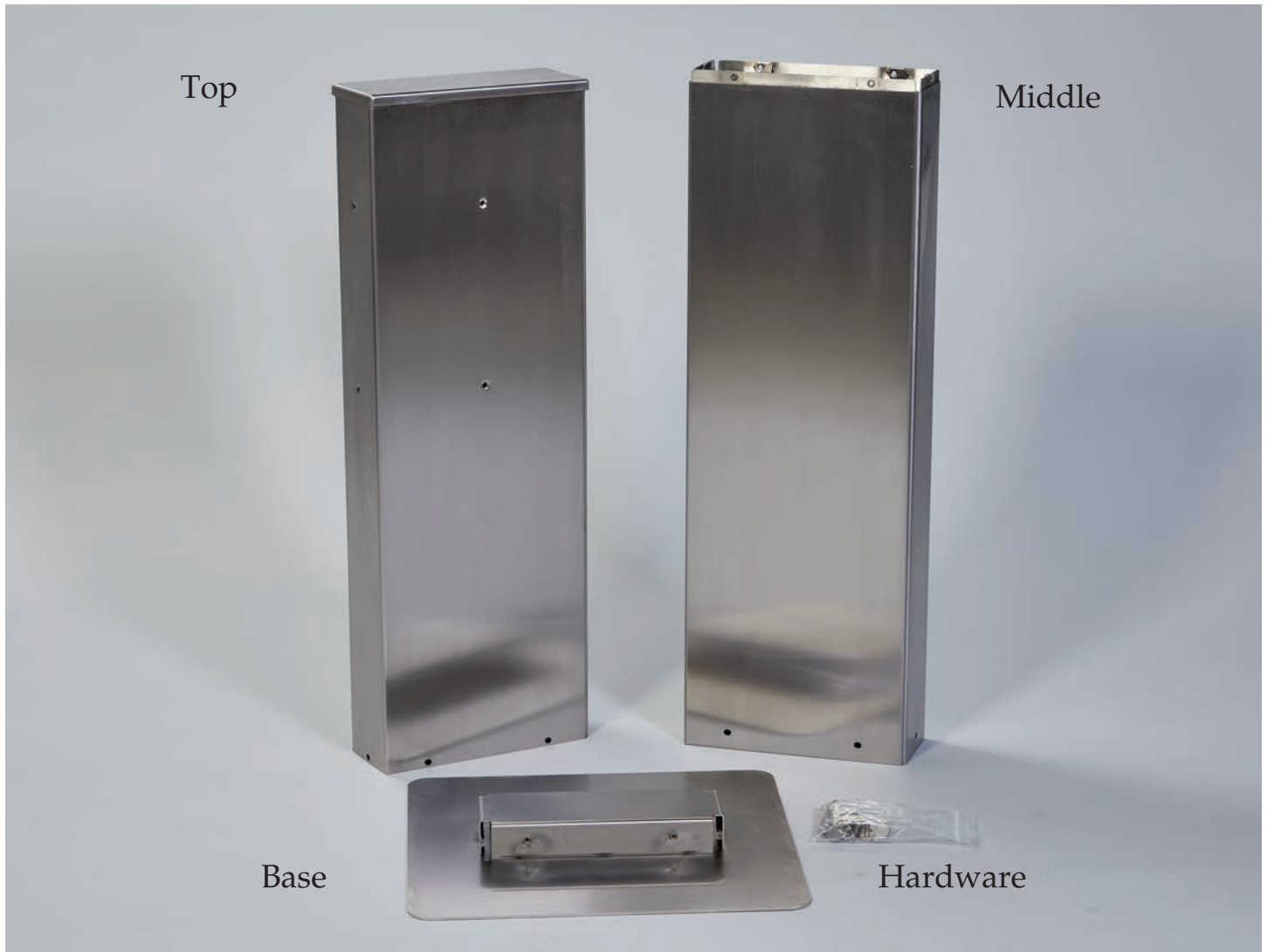

Step 1. Place the middle on the base.

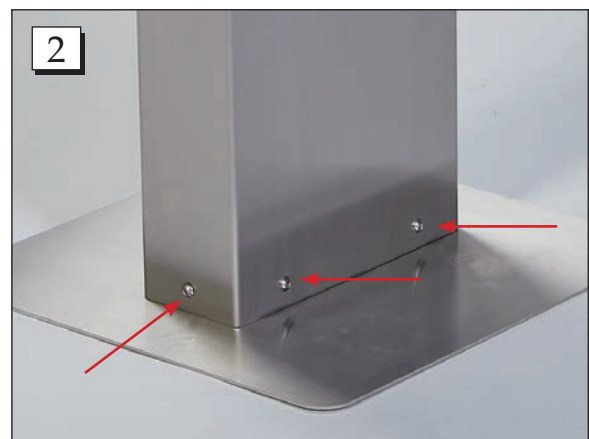


Step 2. Place hex screws on all four sides of the base and tighten with 5/32 hex key (not included).

INSTRUCTIONS FOR


Stainless Steel Stand for HCL® Dispensers (#20947)



Step 3. Place the top on the middle.

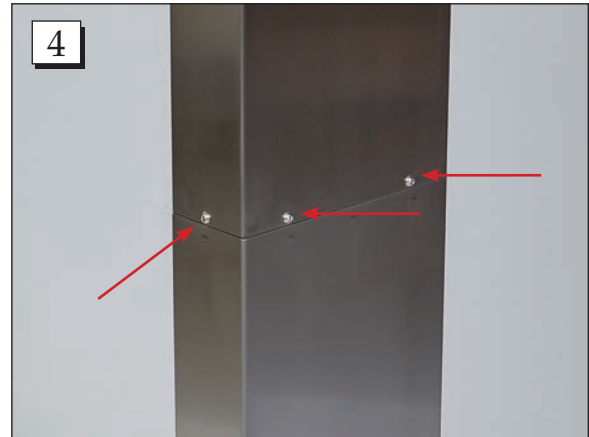

Step 4. Place hex screws on all four sides of the middle and tighten with 5/32 hex key (not included).

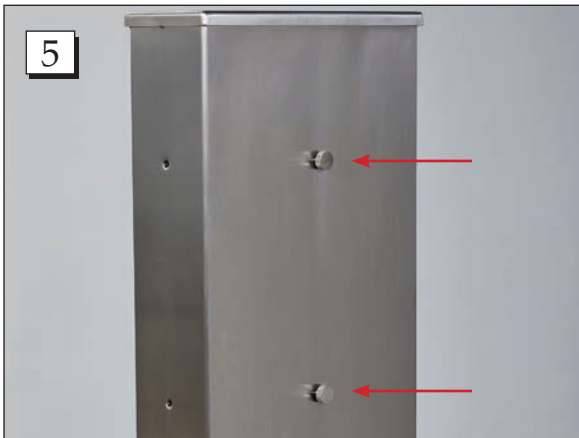

Step 5. Place the thumb screws on all four sides.



Step 6. You are now ready to mount your accessories.