



INSTRUCTIONS FOR Medication Warming Cabinets (Items #4834 & 4858)

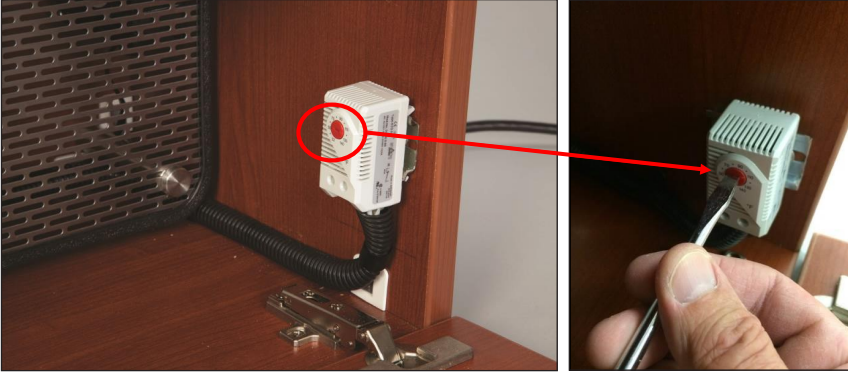

Step 1. Use a straight head screwdriver to preset cabinet to the desired temperature. Close the door and allow the unit to warm for 24 hours.

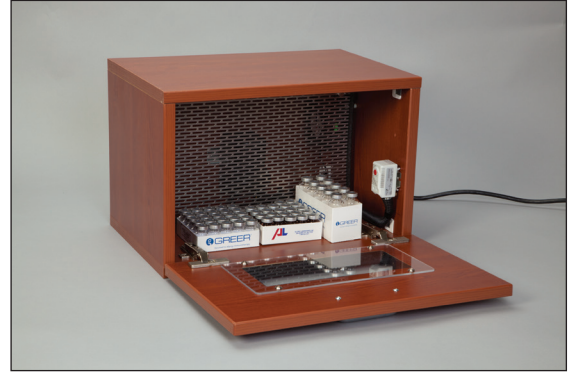

Step 2. After 24 hours place content inside of the warming cabinet and shut the door.

CLEANING INSTRUCTIONS FOR Medication Warming Cabinets (Items #4834 & 4858)

1. Clean with warm water (a microfiber cloth, soft brush or sponge).
2. If needed, use detergent for domestic use (ex. Dawn®, Palmolive® etc.) along with the microfiber cloth.
3. If needed, 50% Alcohol can be used with the microfiber cloth.

It is recommended that you avoid spraying products directly onto the surface. Instead, apply the detergent or 50% Alcohol on a soft damp cloth, clean the surface and rinse it. Avoid using abrasive products, bleach or highly chlorinated products or acids. Do not submerge or immerse the cabinet in water or cleaning products.