

INSTRUCTIONS FOR THE TAMPER EVIDENT SYRINGE CASE (HCL #7226-01)

To Close



The one-piece tamper evident syringe case should have two integral locks built into the case and 10 security seals.



Before closing the lid, make sure that both tabs, which are connected to the box, are projecting out, then fully close the lid.



Bend the tabs downward, and stick them into the slots located directly underneath the tabs.

Sheet#7226-01
Rev.062501

Call Free 1 800 848 1633 or 888 HCL INTL (425 4685) • Fax Free 1 800 447 2923

International Call 740 477 1686 • International Fax 740 477 2923 • Web site <http://www.hcl-intl.com> • E-mail: HCL@hcl-intl.com

INSTRUCTIONS FOR THE TAMPER EVIDENT SYRINGE CASE (HCL #7226)

To Open




To open the syringe case, simply pull up on the tabs.



To Relock



To relock the syringe case, place a security seal tab into the slot, making sure the square tab part is pointing towards the bottom of the box. Once the security seals are inserted into the locks, they cannot be removed without breaking, thus providing evidence of tampering.

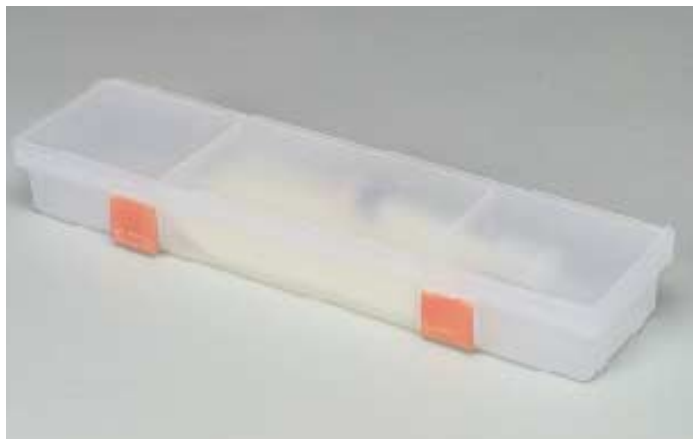

You now have a relocked tamper evident syringe case.

Call Free 1 800 848 1633 or 888 HCL INTL (425 4685) • Fax Free 1 800 447 2923

International Call 740 477 1686 • International Fax 740 477 2923 • Web site <http://www.hcl-intl.com> • E-mail: HCL@hcl-intl.com