

# INSTRUCTIONS FOR Tablet Crusher/Cutter (HCL 7328-01)



1. Turn the top of the cutter/crusher counterclockwise to loosen.



2. Remove the top so that the storage area is accessible.



3. Fill the compartment with meds for storage or remove them for splitting or crushing.



4. Place the top back onto the cutter/crusher and turn it clockwise to secure it.



5. For splitting, flip the lid on the top of the cutter/crusher up by pushing the indented area with your thumb. This allows access to the cutter.



6. Place a tablet into the bell shaped area.

Sheet #7328-01  
Rev.05-23-00

# INSTRUCTIONS FOR Tablet Crusher/Cutter (HCL 7328-01)



7. Allow the shape of the cutting area to center the tablet.



8. Close the lid to split the tablet.



9. Flip the lid open and your tablet should be split in half. If it needs split again, repeat steps 6-8.



10. Remove the cut tablet from the cutting area.



11. To crush your tablet(s), turn the bottom part of the cutter/crusher counterclockwise to remove it.



12. Place your tablet(s) into the bottom section.



13. Replace the bottom section, turning it clockwise to tighten.



14. Tighten the bottom section and rotate it back and forth to crush the tablet



15. Remove the bottom section and administer the crushed tablet.

Call Free 1 800 848 1633 or 888 HCL INTL • Fax Free 1 800 447 2923

International Call 740 477 1686 • International Fax 740 477 2923 • Web site <http://www.hcl-intl.com> • E-mail: [HCL@hcl-intl.com](mailto:HCL@hcl-intl.com)