

INSTRUCTIONS FOR

The Crusher®, Stainless Steel Tablet Crusher for Tall Narrow Cups (#7391-01)



Step 1. Place tablet(s) into a cup.



Step 2. Place the cup into the oval area of the crusher base and cover the tablet(s) with a 2nd cup.



Step 3. Use an up and down tapping motion to crush the tablet(s).



Step 4. Remove the cups from the crusher. The crushed particles are ready to be dispensed.

Call Free 1.800.848.1633 • Fax Free 1.800.447.2923
Web: GoHCL.com • Email: hcl@GoHCL.com