

Instructions for the Pre-Op Mit (HCL #7441-01)



1) Insert either your right or left hand into the opening of the Pre-op Mit.



2) Turn your hand over and remove the paper backing from the top and bottom of palm area on the Mit to expose the adhesive.



3) The entire palm area is now ready to pick up any loose hair or foreign matter on the patient or surrounding prep area.



4) Gently pat the Pre-op Mit over the shaved or prepped area until all loose matter has adhered to the mit. Dispose of mit.



Call Free 1 800 848 1633 or 888 HCL INTL (4685) • Fax Free 1 800 447 2923

International Call 740 477 1686 • International Fax 740 477 2923 • Web site <http://www.hcl-intl.com> • E-mail: HCL@hcl-intl.com