



Instructions for #7500 Security Bag for Blue Bell Crash Cart Box



1. Insert the crash cart box into the opening of the crash cart bag.

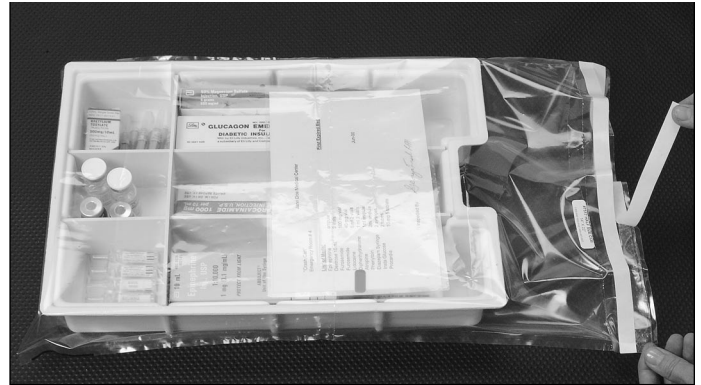

2. Pull off the paper backing to expose the adhesive tape.

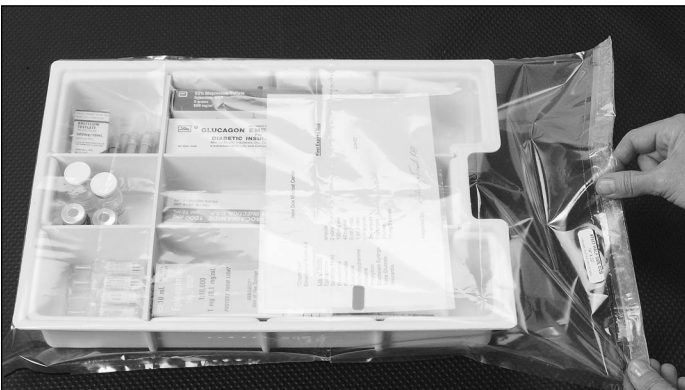

3. Seal the bag by folding over the edge of the bag.



4. To open the bag, remove the box from the cart drawer and tear at the center perforation. Pull one half to the left and the other to the right.



HCL-INTL is a registered trademark of Health Care Logistics, Inc.

Sheet #7500
REV.020802

Call Free 1 800 848 1633 or 888 HCL INTL® (425 4685) • Fax Free 1 800 447 2923
Web Site: www.HealthCareLogistics.com • E-mail: hcl@HealthCareLogistics.com