## Instructions for #7505 Crash Cart Box Security Bag w/ Highlighted "H" Perforations







1. Insert the crash cart box into the opening of the crash cart bag.

2. Pull off both pieces of paper backing to expose the two strips of adhesive.





3. Seal the bag by folding over the edge of the bag until both adhesive strips touch one another.



4. Open by gripping the center part of the bag and tearing at the perforation that reads TEAR HERE. Gripping both sides of the perforation, use one hand to pull the bag up and the other to pull down to tear the top and bottom of the bag.



5. Continue tearing both sides of the bag in order to tear along the top and bottom of the box.



6. When the perforations are torn all the way, the entire box contents are accessible.



Sheet#7505 Rev.071301

HCL-INTL is a registered trademark of Health Care Logistics, Inc.