

Instructions for #7509 Security Bag with Blue Border



1. Insert the crash cart box into the opening of the crash cart bag.



2. Pull off one of the paper backings to expose a strip of adhesive tape.



3. Pull off the other paper backing to expose a second strip of adhesive tape.



4. Seal the bag by folding over the edge of the bag, so that the two strips of adhesive tape are joined together.



5. To open the bag, remove the box from the cart drawer and tear at the center perforation. Pull one half to the left and the other to the right.



HCL-INTL is a registered trademark of Health Care Logistics, Inc.

Rev.071301

Call Free 1 800 848 1633 or 888 HCL INTL® (425 4685) • Fax Free 1 800 447 2923

International Call 740 477 1686 • International Fax 740 477 2923 • Web Site <http://www.hcl-intl.com> • E-mail: HCL@hcl-intl.com