

#7535 Full Size “Adult Only” Amber Crash Cart Bag Instructions



1. Insert the crash cart box into the opening of the crash cart bag.



2. Pull the paper backing off the edge of the bag to expose the adhesive tape.



3. Seal the bag by folding over the edge of the bag.



4. To open the bag, remove from the cart drawer and tear at the center or the end perforation. Pull one half to the left and the other half to the right.



© Health Care Logistics, Inc. 2000

HCL-INTL is a registered trademark of Health Care Logistics, Inc.

Sheet #7535
REV.071701

Call Free 1 800 848 1633 or 888 HCL INTL® (425 4685) • Fax Free 1 800 447 2923
International Call 740 477 1686 • International Fax 740 477 2923 • Web site <http://www.hcl-intl.com> • E-mail: HCL@hcl-intl.com