

#8420 Kango® Stand-up Seat Instructions




1. When you receive your chair it should come with 1 base, 1 seat, 1 extension piece and 1 center column.



2. Slide the extension piece into the base (you can also choose not to use it)



3. Slide the large end of the center column into the extension piece or directly into the base

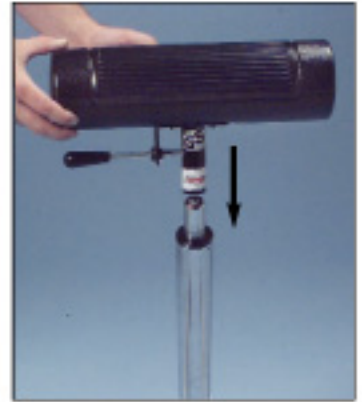

4. Attach the seat to the center column




5. Pull or push the lever under the seat to the desired position.



6. Your chair is now assembled and ready to use.

To use the seat, lean your backside against the round seat. Keep your back straight and legs stretched.

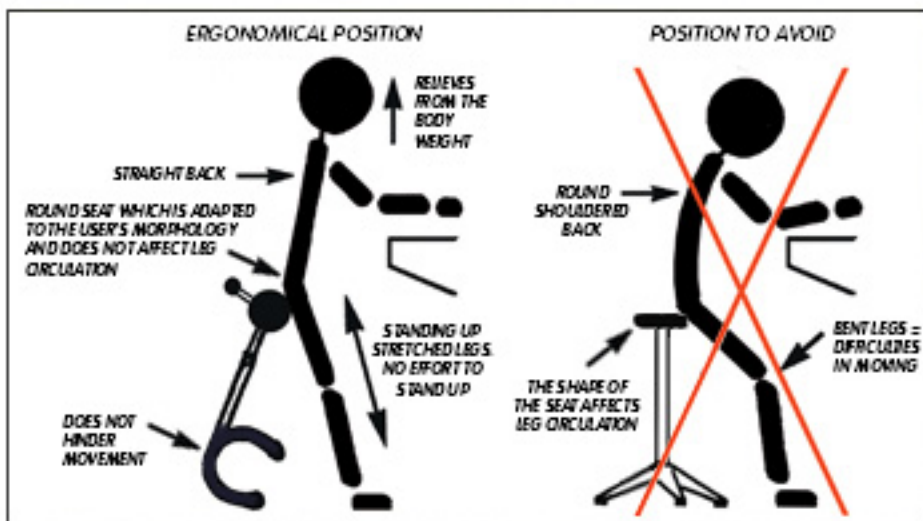

Kango® is a registered trademark of Biens Company

Sheet #8420 REV.081600

HCL-INTL is a registered trademark of Health Care Logistics, Inc.

Call Free 1 800 848 1633 or 888 HCL INTL® (425 4685) • Fax Free 1 800 447 2923

International Call 740 477 1686 • International Fax 740 477 2923 • Web site <http://www.hcl-intl.com> • E-mail: HCL@hcl-intl.com