INSTRUCTIONS FOR #9602-01 2 GALLON SHARPS CONTAINER

tyco / Healthcare

KENDALL

Sharps Disposal Containers with Rotor Openings and AUTODROP* Funnels

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For Sharps Disposal Containers with Rotor Openings and AUTODROP* Funnels.

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English

The use and contents of this container may dictate earlier disposal, but in no event should the container be more than three quarters full. The container should be changed regardless of contents when objects can no longer be dropped freely, without resistance, into the container.

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Your small friend, indeed®
Health Care Logistics®
INC. 

*Trademark of Sherwood Services AG

TYCO HEALTHCARE GROUP LP • MANSFIELD, MA 02048 • MADE IN U.S.A. • © 2001 TYCO HEALTHCARE GROUP LP
ALL RIGHTS RESERVED • 011005 • PRODUCT INFO IN U.S. 1-800-982-9888 • www.kendallhq.com
R9876F

Sheet #9602-01






Rev. 042806

© Health Care Logistics, Inc. 2006 • Printed in the U.S.A.

Call Free: 1 800 848 1633 or 1 888 HCL INTL® (425 4685) Fax Free: 1 800 447 2923
Web Site: www.HealthCareLogistics.com • E-mail: hcl@HealthCareLogistics.com

INSTRUCTIONS FOR #9602-01

2 GALLON SHARPS CONTAINER

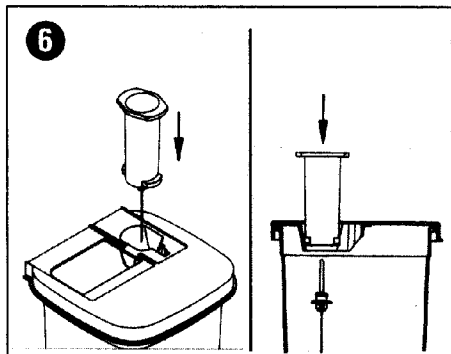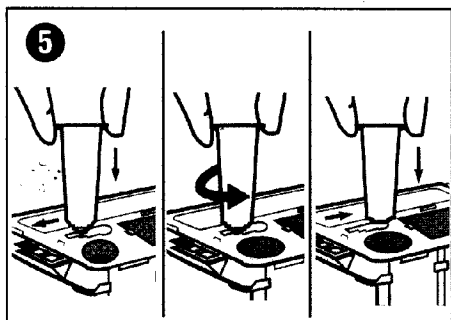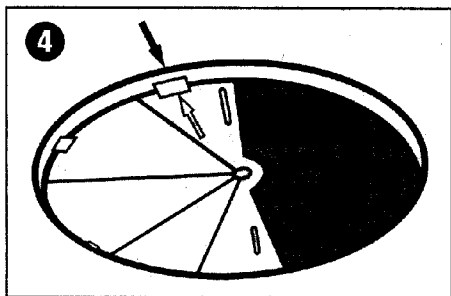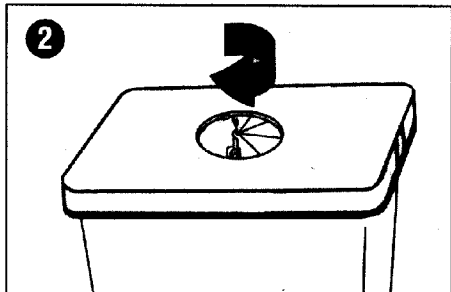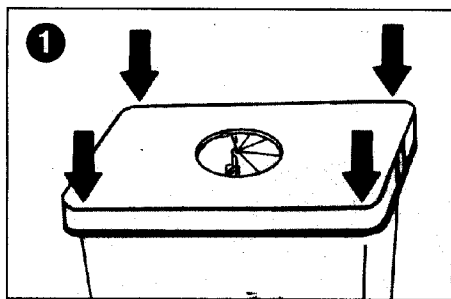English

1. Snap lid on container by pressing down on all four sides. Listen for snaps. Please check that the container has suffered no shipping damage before placing into use.
2. Open rotor by turning clockwise until the opening is the desired size.

TO DISPOSE OF WASTE:

3. Use one hand to drop waste in container. DROP ONE SHARP IN AT A TIME.

FOR FINAL DISPOSAL:

4. TO LOCK THE ROTOR – Turn rotor counterclockwise to the locking position by lining up arrows on rotor and container lid. This is the final lock position for final disposal.

UNWINDER INSTRUCTIONS:

5. TO USE UNWINDER: Needles should not be removed unless required by a specific medical procedure. USE ONE HAND on holder to insert needle into circular opening and slide to proper slot. Unscrew needle and slide holder back to the circular opening allowing needle to fall into container.

FUNNEL INSTRUCTIONS:

6. TO DISPOSE OF NEEDLE – Using one hand, push holder straight down into the funnel of a Kendall Sharps container to release needle and disposal thread.

NOTE: Rotor may be closed temporarily when not in use to prevent accidental contact with contents.

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