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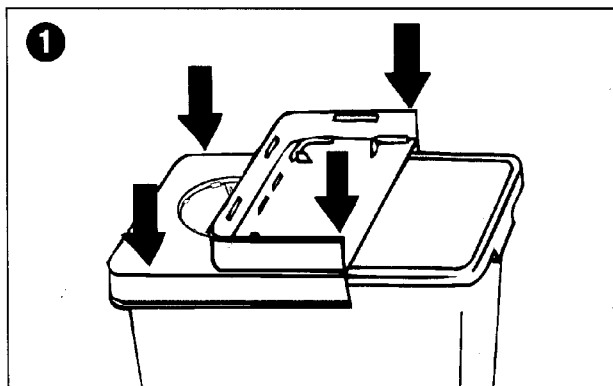
For Containers with Rotor and Hinged Lids

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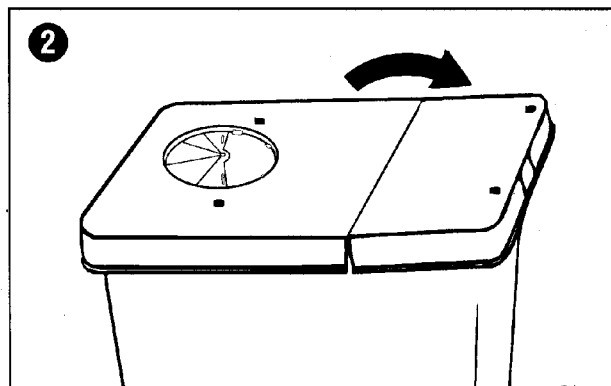
\*Trademark of Sherwood Services AG  
PATENT NO. 4,375,849; 4,779,728; des. 306,509

TYCO HEALTHCARE GROUP LP • MANSFIELD, MA 02048 • MADE IN U.S.A. • © 2000 TYCO HEALTHCARE GROUP LP  
ALL RIGHTS RESERVED • 001023 • PRODUCT INFO IN U.S. 1-800-962-9888 • www.kendallhq.com  
R9875D

Your small friend, indeed®  
**Health Care Logistics**®  
INC.




© Health Care Logistics, Inc. 2006 • Printed in the U.S.A.

Sheet #9603-01  
Rev. 042806

Call Free: 1 800 848 1633 or 1 888 HCL INTL® (425 4685) Fax Free: 1 800 447 2923  
Web Site: www.HealthCareLogistics.com • E-mail: hcl@HealthCareLogistics.com

# INSTRUCTIONS FOR #9603-01 8 GALLON SHARPS CONTAINER

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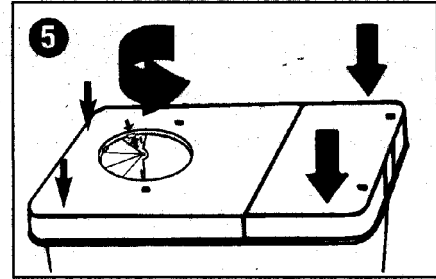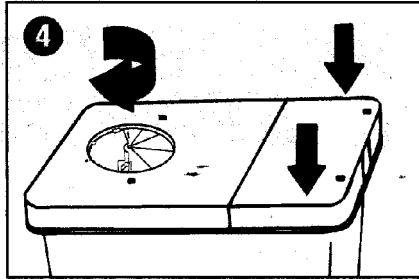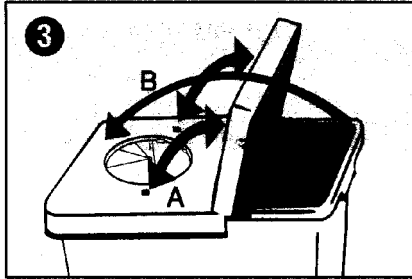• **TO USE HINGED LID** – (A) Open and secure lid by locking lid's lugs in place OR (B) be sure lid is at rest in a fully open position. Use one hand to drop waste in container then temporarily close hinged lid. **DROP ONE SHARP IN AT A TIME.**

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• **TO USE ROTOR OPENING** – Close hinged Lid. Open rotor by turning clockwise until opening is the desired size. **USE ONE HAND TO DROP ONE SHARP IN AT A TIME.**

Rotor may be closed temporarily when not in use to prevent accidental contact with container contents.

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• **FOR FINAL DISPOSAL** – Lock hinged lid by pressing down securely on all edges of the lid. **LISTEN FOR SNAPS.**

**TO LOCK ROTOR** – Turn rotor counterclockwise to the locking position – Lining up arrows.

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• The use and contents of this container may dictate earlier disposal, but in no event should the container be more than three quarters full. The container should be changed regardless of contents when objects can no longer be dropped freely, without resistance, into the container.

• For more guidelines and information on how to use the sharps disposal system, in the U.S. call 1-800-962-9888 for a Sharps Safety Kit.

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Sheet #9603-01  
Rev. 042806


# INSTRUCTIONS FOR #9603-01 8 GALLON SHARPS CONTAINER
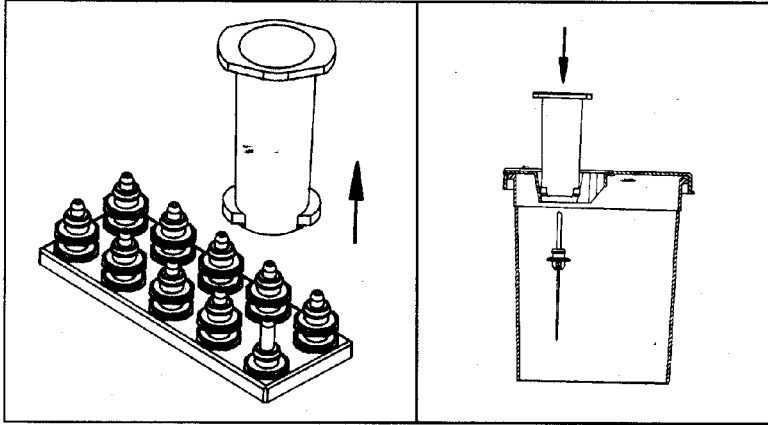
## AUTODROP\* INSTRUCTIONS:

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## INSTRUCCIONES DEL AUTODROP\*:

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### • TO DISPOSE OF NEEDLE:

Using one hand, push holder straight down into the funnel of a KENDALL sharps container to release needle and disposal thread.

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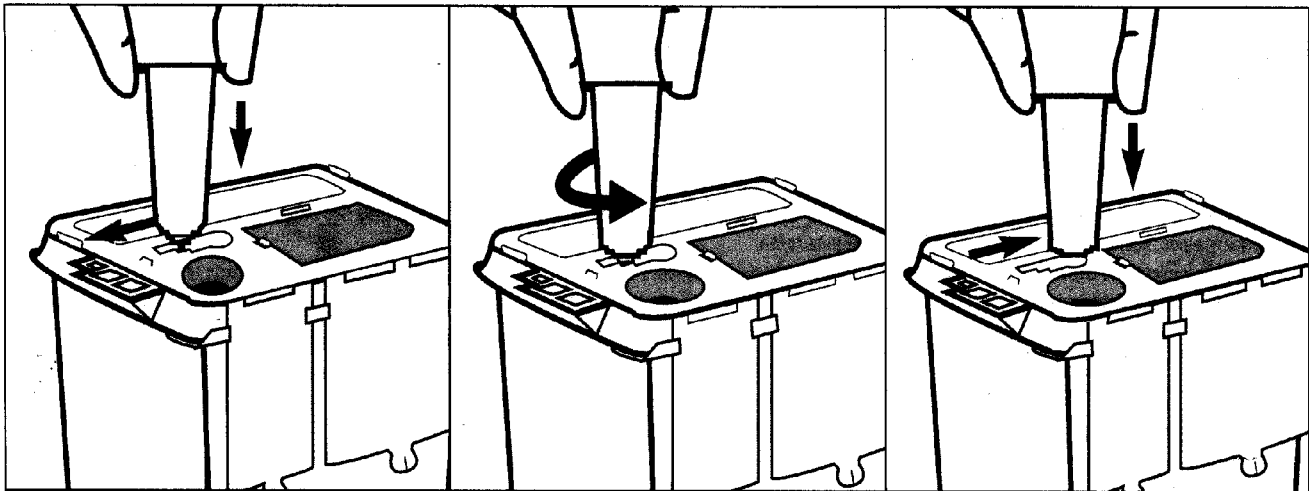
### • TO USE UNWINDER:

Needles should not be removed unless required by a specific medical procedure. USE ONE HAND on holder to insert needle into circular opening and slide to proper slot. Unscrew needle and slide holder back to the circular opening allowing needle to fall into container.

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Sheet #9603-01

Rev. 042806

© Health Care Logistics, Inc. 2006 • Printed in the U.S.A.

Call Free: 1 800 848 1633 or 1 888 HCL INTL® (425 4685) Fax Free: 1 800 447 2923

Web Site: www.HealthCareLogistics.com • E-mail: hcl@HealthCareLogistics.com

# INSTRUCTIONS FOR #9603-01 8 GALLON SHARPS CONTAINER

## Do's and Don'ts for Handling Sharps Disposal Containers

- Do use the proper size container for the items you are disposing of.
- Check fill level to determine if the container should be changed.
- Always use universal precautions when handling any potentially contaminated materials. Wash your hands and wear personal protective equipment.
- Don't allow patients and visitors to treat the sharps container as a waste receptacle or hanger for personal items.
- Don't strike, tap, shake or manipulate a container for any reason.
- Don't carry a used sharps disposal container close to your body. Example: Under your arm or like a book.
- Don't place anything on top of a sharps disposal container or wall enclosure. Example: glove boxes, tissue boxes or used syringes.
- Don't mount or place containers in areas where they may be bumped.

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