INSTRUCTIONS FOR #9604-01 1 QUART SHARPS CONTAINER

tyco | Healthcare

KENDALL

Phlebotomy Sharps Disposal Containers

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







INSTRUCTIONS FOR #9604-01 1 QUART SHARPS CONTAINER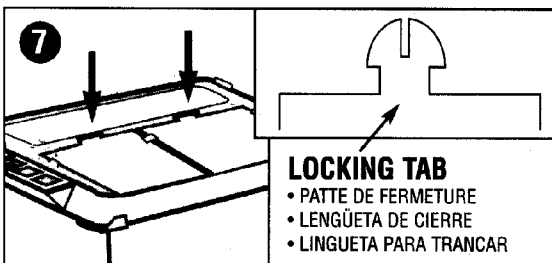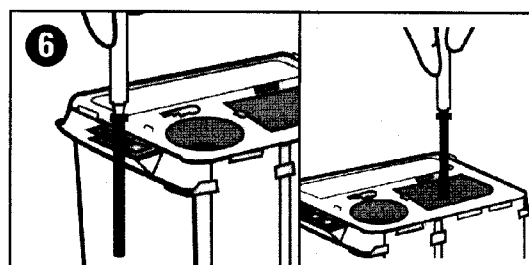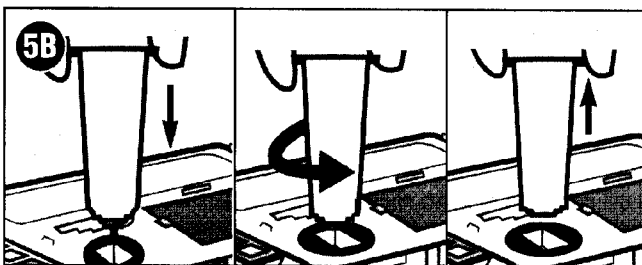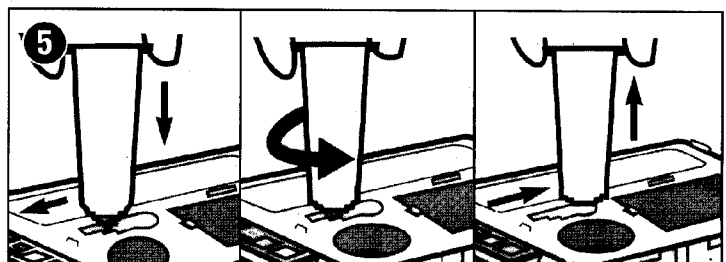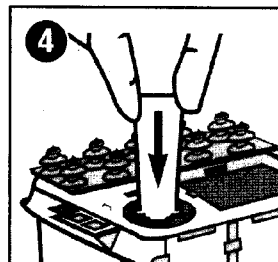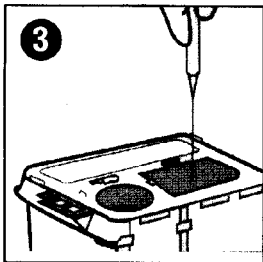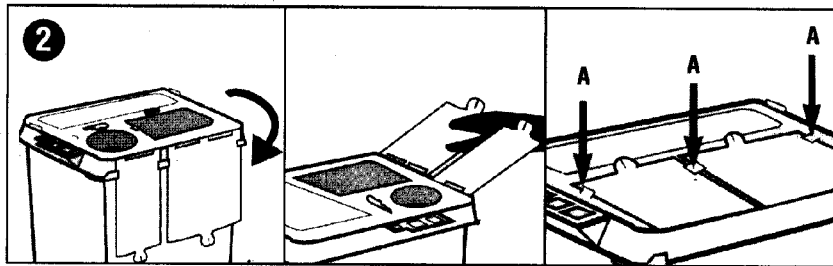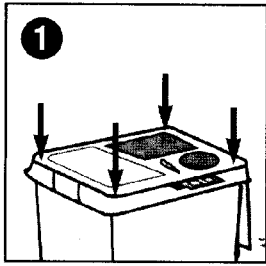

*Trademark of Sherwood Services AG

TYCO HEALTHCARE GROUP LP • MANSFIELD, MA 02048 • MADE IN U.S.A. • © 2001 TYCO HEALTHCARE GROUP LP • ALL RIGHTS RESERVED • 011005 • PRODUCT INFO IN U.S. 1-800-962-9888 • www.kendallhq.com

R98337E

Sheet #9604-01
Rev. 042806

© Health Care Logistics, Inc. 2006 • Printed in the U.S.A.

Call Free: 1 800 848 1633 or 1 888 HCL INTL® (425 4685) Fax Free: 1 800 447 2923
Web Site: www.HealthCareLogistics.com • E-mail: hcl@HealthCareLogistics.com

INSTRUCTIONS FOR #9604-01 1 QUART SHARPS CONTAINER

English

ASSEMBLY AND USAGE INSTRUCTIONS

For Phlebotomy Sharps Disposal Containers

Product Numbers 8900SA, V8900, 8900MW, 8905, 8906, 8908, 8901SA, 8900AC, 8900EC

1. Snap lid on container by pressing down on all four corners. **IMPORTANT – MAKE SURE TOP IS COMPLETELY PUSHED DOWN ONTO CONTAINER.** Check that the container has suffered no shipping damage before placing into use.
(A = LOCKING TABS)
 2. When container is in use, covers should be secured by pressing onto side locking tabs located on the rim of the lid. If desired, separate covers with scissors to allow separate closure. **FOR TEMPORARY CLOSURE –** snap covers down to engage temporary locking tabs. Make sure both sides of covers are engaged. Listen for clicks.
 3. **USE ONE HAND** to drop waste in container. **DO NOT DISPOSE OF MORE THAN ONE SHARP AT A TIME.**
 4. **TO USE AUTODROP* FUNNEL –** USE ONE HAND to disengage needle. Insert needle holder vertically into funnel. Push down until needle is released.
 5. **TO USE UNWINDER –** USE ONE HAND on holder to insert needle into circular opening and slide to proper slot. Unscrew needle and slide holder back to the circular opening allowing needle to fall into container.
 - 5B. **TO USE SAFETY NEEDLE UNWINDER ON 8900EC –** USE ONE HAND on holder to insert Safety Needle into square unwinder opening. Unscrew needle allowing Safety Needle to fall into container.
 6. **TO USE RECAPPER –** Remove cap and place in recapper. After use, USE ONE HAND to put needle in cap and dispose of in the appropriate container.
 7. **FOR FINAL DISPOSAL –** Lock covers by placing locking tabs in slots on the lid.
- The use and contents of this container may dictate earlier disposal, but in no event should the container be more than three quarters full. The container should be changed regardless of contents when objects can no longer be dropped freely, without resistance, into the container.
 - For more guidelines and information on how to use the sharps disposal system, in the U.S. call 1-800-962-9888 for a Sharps Safety Kit.

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Your small friend, indeed®
Health Care Logistics
INC.

INSTRUCTIONS FOR #9604-01 1 QUART SHARPS CONTAINER

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