INSTRUCTIONS FOR #9605-01 2 GALLON CHEMO SAFETY CONTAINER

tyco | Healthcare

KENDALL

Sharps Disposal System

Sharps Disposal System

ASSEMBLY AND USAGE INSTRUCTIONS

For Sharps and Chemo Disposal Containers with Hinged Lids and Snap Cap Openings

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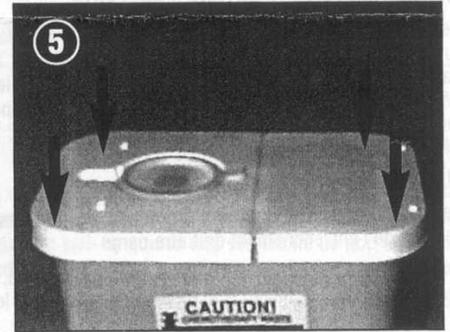
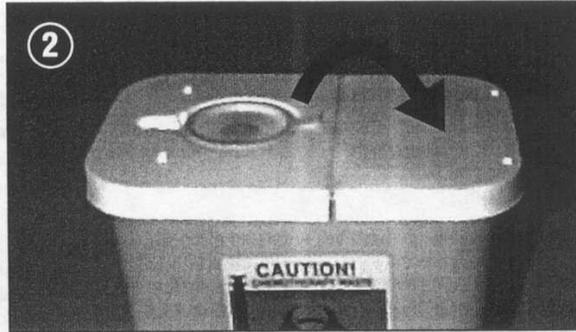
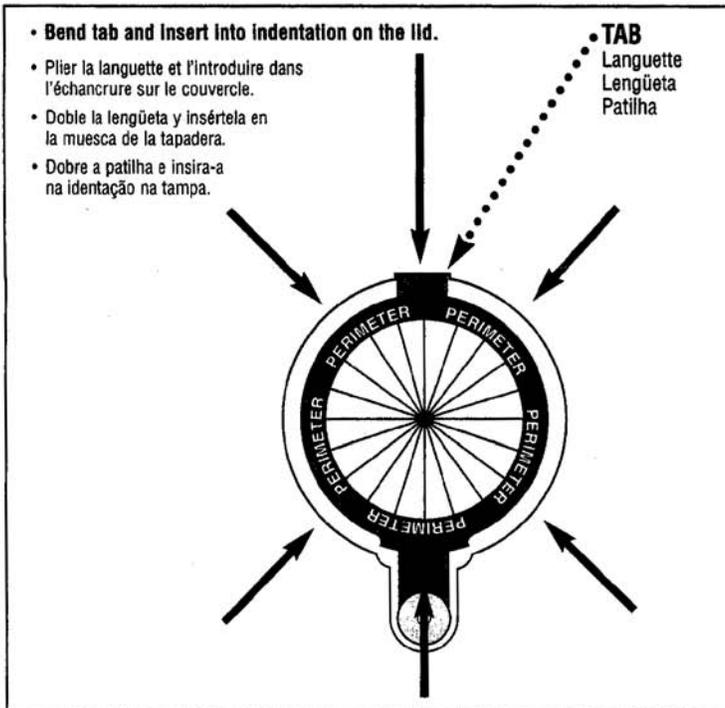
INSTRUCTIONS FOR #9605-01 2 GALLON CHEMO SAFETY CONTAINER

FOR INCINERATION ONLY
POUR INCINÉRATION UNIQUEMENT
SÓLO PARA INCINERACIÓN
APENAS PARA INCINERAÇÃO

TYCO HEALTHCARE GROUP LP • MANSFIELD, MA 02048 • MADE IN U.S.A. • © 2002 TYCO HEALTHCARE GROUP LP
ALL RIGHTS RESERVED • 020429 • PRODUCT INFO IN U.S. 1-800-962-9888 • www.tycohealthcare.com
R9960F

INSTRUCTIONS FOR #9605-01

2 GALLON CHEMO SAFETY CONTAINER

English

ASSEMBLY AND USAGE INSTRUCTIONS

For Sharps and Chemotherapy Disposal Containers with Hinged Lids and Snap Cap Openings

1. Snap lid on container with hinged lid open. Listen for snaps. Please check that the container has suffered no shipping damage before placing into use.
2. Hinged lid may be closed temporarily when not in use by lightly closing door or permanently closed by snapping door all of the way down.

3. TO USE HINGED LID

(A) Open and secure lid by locking lid lugs in place OR (B) be sure lid is at rest in a fully open position. Use one hand to drop waste in container then temporarily close hinged lid. DROP ONLY ONE SHARP IN AT A TIME.

4. TO USE SNAP CAP OPENING

Close hinged lid and pull up on tab to release snap cap. Turn snap cap to the side. To close, place snap cap over opening and apply pressure with thumb along entire perimeter of cap (see step 5). Replace cap when container is not in use. USE ON HAND TO DROP ONLY ONE SHARP IN AT A TIME.

5. FOR FINAL DISPOSAL

Lock door by pressing down securely on all edges of lid. Listen for snaps.

To close Snap Cap – bend tab, insert it into the indentation on the lid and snap cap all the way down.

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