

# Instructions for the Stick-On Biohazard Bags



1. Peel the adhesive strip from the back of the bag.



2. Attach the bag to your desired site (cart, cabinet, wall) with the biohazard symbol facing out so the opening is easily accessible.



3. When the bag is full and removed from its site, peel the adhesive strip on the front of the bag. (the side with the biohazard symbol)



4. Fold the flap over and seal it. Dispose of according to federal, state, and local regulations.

Your small friend, indeed®  
**Health Care Logistics**®  
INC.



**Call Free 1 800 848 1633 or 888 HCL INTL® (425 4685) • Fax Free 1 800 447 2923**

International Call 740 477 1686 • International Fax 740 477 2923 • Web site <http://www.hcl-intl.com> • E-mail: [HCL@hcl-intl.com](mailto:HCL@hcl-intl.com)